

## USA

Trulife  
2010 E. High Street  
Jackson, MI 49203  
USA

Email: [info-usa@trulife.com](mailto:info-usa@trulife.com)  
Tel: 1 800-492-1088  
Fax: 1 800-245-3765

## CANADA

Trulife  
39 East Davis Street  
Trenton, ON K8V 4K8  
Canada

Email: [infocanada@trulife.com](mailto:infocanada@trulife.com)  
Tel: 800-267-2812  
Fax: 613-392-4139

For more information on Trulife products, visit:

[www.trulifebreastcare.com](http://www.trulifebreastcare.com)  
[www.radiantimpressions.com](http://www.radiantimpressions.com)



A3560/1017



## Beautiful Bras for a Natural Beautiful You



Featured by breast cancer survivor models

Trulife bras offer women  
everyday confidence and  
comfort, fashionable  
styling and allow for an  
active lifestyle.

- **Trulife bra styles include:**  
super-soft post-surgery bras and camisoles,  
comfortable and secure everyday bras and  
COOLMAX®-pocketed activity bras.
- **Trulife bras are divided into four categories:**  
Perfectly Seamless, Stylish Fit, Shaped Support and  
Living Well. Always ask to try on multiple styles  
and sizes as it's often difficult to know which style  
or profile will look best on you.
- **All Trulife bras feature pockets on both sides  
and are made from breathable soft mesh  
or COOLMAX®.**  
Some bras feature: beautiful embroidery, padded  
shoulder strap, delicate lace trim, front-closure  
styling and supportive underwire.

Ask your Trulife retailer to see a selection of our  
beautiful, comfortable, functional bras.

## How to tell if you have a perfect, flawless fit?

The characteristics of a properly fit bra are:

- **Bra should be centered over your sternum**  
– When a breast form is placed in the cup of the  
bra, the bra should be centered over your sternum.  
It's normal for some shifting to one side to occur.  
Major shifting is the result of a cup that is too small.
- **Bra should “tack” or touch your sternum**  
– If the center of your bra does not tack or touch  
your sternum, the cup size is too small.
- **Bra cup should contain all of your breast tissue**  
– If the cup size is too small, tissue will “spill” out  
and cause an unsightly appearance and profile. If  
the cup is too small, increase the cup size by one  
cup and decrease the band size.
- **Bra should be snug at your underarm and  
bottom band** – Support of a bra comes from the  
band, not the straps. This is a big misconception.  
Many women believe a loose bra will be more  
comfortable. Nothing could be farther from the  
truth. A snug band will provide optimal support  
and comfort. The more snug the band, the more  
comfortable the fit. Don't believe it...experience it  
for yourself and feel the difference!

What are you waiting for?  
Contact your Trulife professional fitter today to view our bra collection.  
She'll be delighted to show you all of your options and provide you with  
the perfect fit that you deserve.



## Shaped Support

297



202



### 297 Rose

#### Full Support Softcup

COLORS: White, Nude, Black

SIZES: 34-42B, 34-46C, 36-48D, 36-48DD, 36-48DDD

### 202 Mandy

#### Three-Quarter Length Posture Support Softcup

COLORS: White, Nude

SIZES: 36-38A, 36-42B, 36-46C, 38-48D, 38-48DD

## Stylish Fit

### 4019 Jessica

#### Cami Style Lace Accent Softcup

COLORS: White, Nude, Black, Powder Pink, Amethyst

SIZES: 34-42A, 34-44B, 34-48C, 34-48D, 36-48DD, 36-46DDD

### 212 Bethany

#### Front & Back Closure Lace Accent Softcup

COLORS: White, Nude

SIZES: 34-42B, 34-44C, 34-44D, 34-44DD

210



### 210 Barbara

#### Lace Accent Softcup

CORE COLORS: White, Nude, Black, Latte

SIZES: 32-42A, 32-52B, 34-52C, 34-52D, 34-48DD, 34-48DDD

FASHION COLORS: Amethyst, Sandstone, Midnight Blue, Powder Pink

SIZES: 34-42A, 34-44B, 34-48C, 34-48D, 36-48DD, 36-46DDD

4019



212



## Perfectly Seamless



4033



4028



### 4033 Daniela

#### Multiway Seamless Molded Softcup

COLORS: White, Nude, Black

SIZES: 34-42A, 34-44B, 34-42C, 34-40D, 34-38DD

### 4028 Amelia

#### Seamless Lace Accent Softcup

COLORS: White, Latte, Black

SIZES: 34-40A, 34-42B, 34-42C, 34-42D

4013



### 4013 Alexandra

#### Seamless Molded Softcup

COLORS: White, Latte, Black, Amethyst, Sandstone

SIZES: 34-42A, 34-44B, 34-42C, 34-40D, 34-38DD

## Living Well

330



4014



### 330 Sophia

#### Activity Softcup

COLORS: White, Grey, Black

SIZES: 34, 36, 38, 40, 42, 44 A/B and C/D

### 4014 Ella

#### Seamless Cotton/Activity Softcup

COLORS: Black, Latte

SIZES: 34-42A, 34-42B, 34-42C, 34-42D

### 603 Jennifer

#### Step-In Camisole with Drain Pouches

COLORS: White, Ivory, Black

SIZES: S, M, L, XL, 2XL, 3XL

603

